LEVORAG

EMULGEL

CE
1936

it LEVORAG EMULGEL

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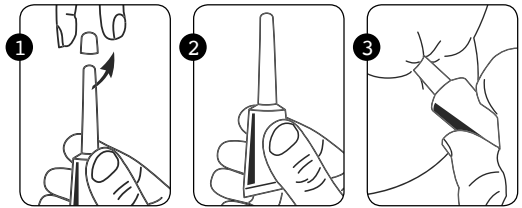
en LEVORAG EMULGEL

for the treatment of anal fissures.

Intended use: The medical device LEVORAG – EmulGel is used for the symptomatic and asymptomatic treatment of anal fissures.

Description: The medical device LEVORAG – EmulGel consists of a gel cream in 3.5ml single-dose tubes with built-in applicator. The device provides emollient, lubricating, film-forming, soothing and protective properties. The device creates a film over the anal fissures making it possible to maintain a microenvironment protected from external microorganisms. The oily part of the formulation helps to lubricate the area being treated by facilitating the expulsion of faeces and thus preventing the formation of further microlesions. The device helps relieve pain upon defecation, prevent the formation of further fissures, and protect the treated area by creating an environment that is conducive to healing of fissures and non-conductive to bacterial attachment. The natural ingredients it contains contribute to a rapid well-being and recovery of optimal physiological conditions. It contains no cortisone, lidocaine, fragrance, dyes, or nickel.

Contraindications: Do not use in the case of known hypersensitivity to any component of the product. If you experience unexpected reactions stop treatment and consult a doctor. Do not apply the device to areas treated simultaneously with other ointments or disinfectants. Improper use of the device may result in delayed tissue repair by preventing the physiological healing process. It is not recommended for use in patients with:

- Expulsive constipation requiring prolonged straining or manual manoeuvring during evacuation.

- Presence of faecal incontinence.

Instructions for use and dosage: Thoroughly clean the area to be treated; the use of Delicin disinfecting wipes is recommended.

1. Remove the cap on the end of the applicator;
2. grip the tube as shown,
3. insert the applicator gently into the anal orifice, **taking care not to fully insert the cannula**, and press the container thoroughly to dispense the entire contents of the tube.

A single-dose container is for one application. Apply twice a day, for at least 20 consecutive days, unless otherwise advised by your doctor; preferably morning and evening, if possible after bowel movements.

Disposal: Dispose of the containers after use in conformity with the current laws and regulations.

Warnings and precautions for use: In the event of adverse reactions, stop treatment and consult your doctor. Avoid the use of disinfectants because they have a negative effect on the wound healing process. Currently there is no evidence of any contraindications for the use of any of the components of LEVORAG – EmulGel during pregnancy or breast-feeding. However, for caution's sake it is recommended for use under the direct supervision of your physician. Avoid contact with eyes. Keep out of the reach of children. If symptoms persist after treatment, please consult your doctor.

Storage and expiry date: Store in a cool, dry place, away from heat sources.

The expiry date is stated on the box and on each individual single-dose, do not use the product after the indicated date.

Explanation of symbols: See bottom of box.

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