# THDCREAM

it THDCREAM

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en THDCREAM

EmulGel for the treatment of haemorrhoids.

**Intended use:** The medical device THDCream is an adjuvant for treatment of haemorrhoidal disease and related symptoms, such as pain, burning and itching sensation. It promotes the reduction of inflammation and restoration of optimal physiological conditions, helping repair the anal epithelium. **Description:** The medical device THDCream consists of a gel cream in tubes and two tube applicators, contained in each pack. The cream has hydrating, emollient, lubricating, filmogenic, soothing and protective properties. The cream creates a film on the haemorrhoids which protects the treated zone and creates a favourable environment for healing and hostile to bacterial attack. The oily part of the formulation helps lubricate the area being treated and facilitates the expulsion of faeces, thus reducing stress on the inflamed area, further contributing to pain reduction during defecation. The natural active ingredients contribute to a rapid return to well-being, helping restore optimal physiological conditions, and also contain flavonoids whose anti-oxidant action helps improve micro-circulation in the interested area. Does not contain: cortisone, local anesthetics, colorant and nickel. **Composition:** See the pack. **Contraindications:** Do not use in case of known hypersensitivity to one or more components of the product. Use, especially prolonged, of the products for topic use, can give rise to sensitivity phenomena. If any unexpected reactions to the medication occur, stop treatment and consult your doctor. **Warnings and Precautions for Use:** Inform your doctor of any contemporary assumption of medications or other products, including those which require no prescription. Currently there are no known contraindications against use of the components contained in THDCream during pregnancy or breast-feeding, however for caution's sake its use is not recommended during pregnancy or breast-feeding. Avoid contact with the eyes. Do not swallow. Keep out of the reach of children. If symptoms persist after treatment, please consult your doctor. Do not use if the original package appears damaged or open. **Dosage and Instructions for use:** One application per day is recommended, possibly following a bowel movement.

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It is recommended to apply the cream one more time if a bowel movement occurs after its application. Carefully cleanse the area to be treated; the use of Deliclin soap is recommended as it is specifically developed for the perianal area. Remove the cap and screw in the special applicator cannula provided with its protection cap. Remove the cap and squeeze out a small quantity of product which when applied on the tip of the applicator will have a lubricating function. Insert in the anus and press the tube to dispense a sufficient quantity of product. Extract the tube and apply a small quantity of product on the external anal skin. Carefully clean the cannula before inserting the protection cap. Alternatively, apply the product using the fingers in a sufficient quantity to abundantly cover the treated area and gently massage until the product has been absorbed. Carefully wash hands after use. It is recommended not to use the product for more than 30 consecutive days. **Storage and expiry date:** Store in a cool dry place, away from sources of heat. The expiry date is indicated on the box and the tube and refers to the product when properly stored in an intact package. Do not use beyond the expiry date. Once opened the product should be used within 8 months. **Disposal:** Dispose of the containers following use in compliance with the existing laws. Do not litter. **Explanation of symbols:** See instructions on the bottom.

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