



## THD FUSION FLEX 6" TIPS

### THD 14-1036 FUSION FLEX 6" BALL 3MM P/N 8002216



The Smoke-Evac Fusion Flexible 6" 3mm Ball probe is used to seal blood vessels during a procedure through electrothermal coagulation. This probe is also used to treat anal dysplasia in High Resolution Anoscopy (HRA) cases. This flexible probe allows for providers to achieve virtually infinite angles of treatment. Harmful surgical smoke will be suctioned through the tip of the electrode. As char accumulates from desiccated tissue, clean off with moistened gauze. This probe must be used with the Smoke-Evac Fusion Hand Piece (800206) or a specialty tubing kit (800212). For best results, use with THD Light-Scope Procto HRA (800191) and bend probe ~2cm from the hand piece to desired angle. Always activate smoke-evacuator during treatment to ensure optimal visibility and safety.

### THD 14-1066 FLEX 6" BLADE P/N 800219



The Smoke-Evac Fusion Flexible 6" Blade is a Teflon-coated, stainless steel blade that is used to ablate lesions in High Resolution Anoscopy (HRA) cases using electrothermal fulguration. This flexible probe allows for providers to achieve virtually infinite angles of treatment. Harmful surgical smoke will be suctioned through the tip of the electrode. As char accumulates from desiccated tissue, clean off with moistened gauze. This probe must be used with the Smoke-Evac Fusion Hand Piece (800206) or a specialty tubing kit (800212). For best results, use with THD Light-Scope Procto HRA (800191) and bend probe ~2cm from the hand piece to desired angle. Always activate smoke-evacuator during treatment to ensure optimal visibility and safety.



## **THD 14-1046 FUSION FLEX 6" BALL 5MM P/N 800217**



The Smoke-Evac Fusion Flexible 6" 5mm Ball is used to seal larger blood vessels using electrothermal coagulation. This flexible probe can also be used to treat more extensive cases of anal dysplasia in High Resolution Anoscopy (HRA) cases. This flexible probe allows for providers to achieve virtually infinite angles of treatment. Harmful surgical smoke will be suctioned through the tip of the electrode. As char accumulates from desiccated tissue, clean off with moistened gauze. This probe must be used with the Smoke-Evac Fusion Hand Piece (800206) or a specialty tubing kit (800212). For best results, use with THD Light-Scope Procto HRA (800191) and bend probe ~2cm from the hand piece to desired angle. Always activate smoke-evacuator during treatment to ensure optimal visibility and safety.

## **THD 14-1045T FUSION FLEX 6" SUCT COAG TIP**



### **P/N 800220**

The Smoke-Evac Fusion 6" Flexible Suction Coag Tip is a Teflon-Coated tip that is used to suction both smoke and fluids while sealing blood vessels to allow for faster hemostasis. This flexible probe allows providers to achieve virtually infinite angles of treatment. Harmful surgical smoke and fluids will be suctioned through the tip of the electrode. If using an electrosurgical generator, this probe must be used with the Smoke-Evac Fusion Hand Piece (800206) and Tissue Catch Tubing Kit (800208). Recommended for use with THD Light-Scope Procto HRA (800191).



## THD 14-1056 FUSION FLEX 6" NEEDLE TIP P/N 800218



The Smoke-Evac Fusion Flexible 6" Needle Tip is used for removal of condylomas in High Resolution Anoscopy (HRA) cases. The fine tip allows for cutting through tissue even when using a high frequency desiccator. This flexible probe allows for providers to achieve virtually infinite angles of treatment. Harmful surgical smoke will be suctioned through the tip of the electrode. As char accumulates from desiccated tissue, clean off with moistened gauze. This probe must be used with the Smoke-Evac Fusion Hand Piece (800206) or a specialty tubing kit (800212). For best results, use with THD Light-Scope Procto HRA (800191) and bend probe ~2cm from the hand piece to desired angle. Always activate smoke-evacuator during treatment to ensure optimal visibility and safety.

| THD FUSION FLEX 6" TIPS |        |   |
|-------------------------|--------|---|
| Q.ty                    | Code   | Denomination                              |
| 1                       | 800216 | THD 14-1036 FUSION FLEX 6" BALL 3MM       |
| 1                       | 800219 | THD 14-1066 FLEX 6" BLADE                 |
| 1                       | 800217 | THD 14-1046 FUSION FLEX 6" BALL 5MM       |
| 1                       | 800220 | THD 14-1045T FUSION FLEX 6" SUCT COAG TIP |
| 1                       | 800218 | THD 14-1056 FUSION FLEX 6" NEEDLE TIP     |

| CONNECTED DEVICES |        |  |
|-------------------|--------|--|
| Q.ty              | Code   | Denomination                           |
| 1                 | 800181 | THD SMOKE EVACUATOR SE02-220           |
| 1                 | 800231 | THD DERM A942 HIGH FREQ.               |
| 1                 | 800182 | THD A1250 ESU (ELECTRIC SURGICAL UNIT) |



## THD FUSION 2" TIPS

### THD 14-1022 FUSION 2" BALL 5MM P/N 800224



The Smoke-Evac Fusion 2" 5mm Ball Tip is used to seal larger blood vessels for perianal treatments using electrothermal coagulation. This probe can be used to treat more extensive cases of perianal dysplasia when more blood is present in High Resolution Anoscopy (HRA) cases. Harmful surgical smoke will be suctioned through the tip of the electrode. As char accumulates from desiccated tissue, clean off with moistened gauze. This probe must be used with the Smoke-Evac Fusion Hand Piece (800206) or specialty tubing kit (800212).

### THD 14-1002PA FUSION 2" PLASTIC BLADE P/N 800222



The Smoke-Evac Fusion 2" Plastic Blade is a Teflon-coated, stainless steel blade that is used to ablate perianal lesions in High Resolution Anoscopy (HRA) cases using electrothermal fulguration. As char accumulates from desiccated tissue, clean off with moistened gauze. This probe must be used with the Smoke-Evac Fusion Hand Piece (800206) or a specialty tubing kit (800212).



## THD 14-1012 FUSION 2'' NEEDLE TIP P/N 800223

The Smoke-Evac Fusion 2'' Needle Tip is used for removal of external condylomas in High Resolution Anoscopy (HRA) cases. The fine tip allows for cutting through tissue even when using a high frequency desiccator. As char accumulates from desiccated tissue, clean off with moistened gauze. This probe must be used with the Smoke-Evac Fusion Hand Piece (800206) or a specialty tubing kit (800212).

| THD FUSION 2'' TIPS |        |  |
|---------------------|--------|--|
| Q.ty                | Code   | Denomination                           |
| 1                   | 800224 | THD 14-1022 FUSION 2'' BALL 5MM        |
| 1                   | 800222 | THD 14-1002PA FUSION 2'' PLASTIC BLADE |
| 1                   | 800223 | THD 14-1012 FUSION 2'' NEEDLE TIP      |

| CONNECTED DEVICES |        |  |
|-------------------|--------|--|
| Q.ty              | Code   | Denomination                           |
| 1                 | 800181 | THD SMOKE EVACUATOR SE02-220           |
| 1                 | 800231 | THD DERM A942 HIGH FREQ.               |
| 1                 | 800182 | THD A1250 ESU (ELECTRIC SURGICAL UNIT) |